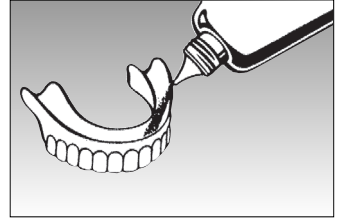
Adhesive Powder
Extra-Strong

Polvo adhesivo
Extra-Fuerte

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GB **Protifix® Adhesive Powder**

Instructions for use. Please read carefully!

Application: Adhesive powder for dentures.

Dosage and mode of application: Sprinkle **Protifix® Adhesive Powder Extra-Strong** thinly and uniformly onto the wet denture. The denture fits securely immediately after being put into position and remains so for hour after hour. In case of the premature loosening of the dentures (because of drinking large quantities of liquid, for example), simply sprinkle on a little more **Protifix® Adhesive Powder Extra-Strong**. Your denture will then fit securely again. Any residue of powder can be removed easily from the denture under running water with the help of a toothbrush. Stubborn residue can be removed with a cotton-wool ball soaked in edible oil. To prevent excess powder from escaping, please do not squeeze container. Hold at an angle and tap gently with the index finger. This will give the correct amount.

Contraindications: Hypersensitivity to the ingredient.

Side effects: Isolated cases of hypersensitivity reactions in the mucous membranes of the mouth and even systemically have been observed. If you suspect the presence of a hypersensitivity reaction, please contact your dentist.

Please note the following important advice: To be used only on healthy mucous membranes. Please clarify this with your dentist. Keep out of reach of children. Do not use after the expiry date. The use of adhesive powder should only be temporary. Please check the adequate form of your dentures regularly.

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Queisser
 PHARMA
 D-24914 Flensburg
 
 Made in Germany

GB

Firm hold for dentures with **Protefix**[®]

Dentures should not prevent you from living life to the full, because this is a justifiable wish of all denture-wearers. There is almost always a solution to denture-wearers' problems. Such problems can occur, for example, when too little saliva is being produced or in the presence of jaw problems. The same applies to temporary or new dentures. As a result, the denture does not fit properly and is painful to wear. Not only that, but you also feel ill at ease in company. **Protefix**[®] Adhesives provide you with quick and reliable remedies to your denture problems.

The complete range of adhesive and care products for dentures:

- Protefix**[®] Adhesive Cushion for Upper and Lower Dentures for jaw problems and interim interfacing, prevents sores, provides firm denture hold in cases of severe jaw degeneration, with wet adhesion
- Protefix**[®] Adhesive Cream Extra-Strong with wet adhesion and long-lasting effectiveness, can be applied straight to the wet denture, when a large amount of saliva is being produced, tasteless
- Protefix**[®] Adhesive Powder Extra-Strong 100 % natural sodium alginate, provides firm hold when too little saliva is being produced, with wet adhesion, tasteless
- Protefix**[®] Active Cleanser with Active Oxygen High cleaning power. Prevents bad breath. Freshness and cleanliness all day long.

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