
Instructions for use.
Please read carefully!

Firm hold for dentures. **Protefix®**

Protefix® Adhesive Cream with Aloe Vera

Extra strong • Effective on wet denture • Soft care for mucosa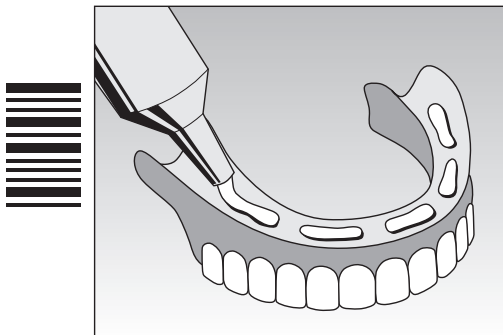

Application

Protefix® Adhesive Cream with Aloe Vera for dentures.

Dosage and mode of application


Daily dosage: up to 0.8 g adhesive cream. Clean the dentures of any residue and dab **Protefix® Adhesive Cream with Aloe Vera** on the wet denture (in the upper or lower denture recess that goes next to the jaw). Press the tube in the lower third near the opening. If necessary, you may also apply a line of **Protefix® Adhesive Cream with Aloe Vera** to the back of the upper plate. You should apply **Protefix® Adhesive Cream with Aloe Vera** sparingly because any excess can impair the adhesion of your denture. Then fit the denture and hold it firmly in place for a few seconds. Wait 5 minutes before eating. The adhesive cream does not noticeably change the position of the denture. After applying the cream, seal the tube carefully, taking care to ensure that the opening remains dry and clean because any ingress of liquid may cause the adhesive cream to harden. Remove any residue of adhesive cream with a cotton-wool ball soaked in edible oil. Under certain circumstances oil may appear, which has no effect whatsoever on the cream's particularly good adhesive force. Please just remove the oil from the dentures. The use of adhesive cream should only be temporary.

Contraindications

Hypersensitivity to one of the ingredients.

Side effects

In isolated cases, hypersensitivity reactions in the mucous membranes of the mouth or even systemically cannot be ruled out. If you suspect the presence of a hypersensitivity reaction, please contact your dentist.

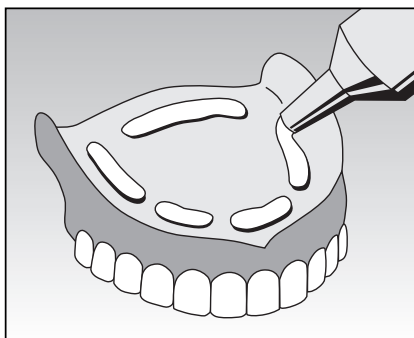Please note the following important advice:

- To be used only on healthy mucous membranes.
- Keep out of reach of children.
- Do not use after the expiry date.

Composition

Mixture of sodium and calcium salts of the copolymer of methyl vinyl ether and maleic acid anhydride, carboxymethyl cellulose, paraffin, vaseline, Aloe Vera Barbadosis Miller, silica, menthol, azorubin, p-hydroxy-benzoic acid methyl ester. – Does not contain zinc.

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Sunstar Americas Inc.,
Laboratorios Distrosur, S.A.