

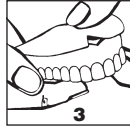
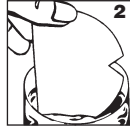
Upper denture Prótesis del maxilar superior 用于上颌义齿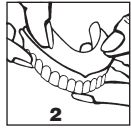

Lower denture Prótesis del maxilar inferior 用于下颌义齿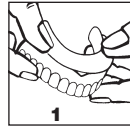

GB Protifix Adhesive Cushions

Instructions for use. Please read carefully!

Application: Adhesive for dentures.

Dosage and mode of application: (see diagrams above)

Upper denture: 1. **Check size:** For smaller dentures, trim the adhesive cushion to fit with scissors. 2. **Moisten cushion:** Dip the adhesive cushion into lukewarm water for approx. 10 seconds and allow to drain. 3. **Apply cushion:** Place either side of the cushion on the moistened denture. The rearmost 2 mm of the denture must remain uncovered. 4. **Press on cushion:** Press the cushion onto the dentures starting in the centre and working outwards until the cushion is smooth all over.

Lower denture: 1. For small dentures, trim the adhesive cushion to fit. Moisten the cushion and apply as described for the upper denture. 2. **Press on cushion:** Press the cushion into the jaw recess of the dentures using the index finger (starting in the centre and working outwards until the cushion is smooth all over). **Tip:** Should the adhesion begin to diminish, please remove the dentures and sprinkle the cushion with **Protifix** Adhesive Powder.

Contraindications: Hypersensitivity to one of the ingredients.

Side effects: In isolated cases, hypersensitivity reactions in the mucous membranes of the mouth or even systemically cannot be ruled out. If you suspect the presence of a hypersensitivity reaction, please contact your dentist.

Please note the following important advice: For full dentures only. Not for partial dentures in conjunction with attachments or tapered crowns. To be used only on healthy mucous membranes. Please clarify this with your dentist. For hygiene reasons, the **Protifix** Adhesive Cushion must be changed at least once a day. Store **Protifix** Adhesive Cushions as dry as possible. Do not use after the expiry date. Keep out of reach of children. The use of adhesive cushions should only be temporary. Please check the adequate form of your dentures regularly.

Composition: fleece made of sodium alginate, viscose fibre, polypropylene fibre.

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Queisser
 PHARMA
 D-24914 Flensburg
 
 Made in Germany

GB Firm hold for dentures with Protefix

Dentures should not prevent you from living life to the full, because this is a justifiable wish of all denture-wearers. There is almost always a solution to denture-wearers' problems. Such problems can occur, for example, when too little saliva is being produced or in the presence of jaw problems. The same applies to temporary or new dentures. As a result, the denture does not fit properly and is painful to wear. Not only that, but you also feel ill at ease in company. **Protefix** Adhesives provide you with quick and reliable remedies to your denture problems.

The complete range of adhesive and care products for dentures:

- Protefix** Adhesive Cushion for Upper and Lower Dentures for jaw problems and interim interfacing, prevents sores, provides firm denture hold in cases of severe jaw degeneration, with wet adhesion
- Protefix** Adhesive Cream Extra-Strong with wet adhesion and long-lasting effectiveness, can be applied straight to the wet denture, when a large amount of saliva is being produced, tasteless
- Protefix** Adhesive Powder Extra-Strong 100 % natural sodium alginate, provides firm hold when too little saliva is being produced, with wet adhesion, tasteless
- Protefix** Active Cleanser with Active Oxygen High cleaning power. Prevents bad breath. Freshness and cleanliness all day long.

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